


Health Monitor GMON

„INDICATE Health Risks – RATE professionally – REACT prophylactically”

Manual GMON – TANITA ViScan

Taking a new reading

By clicking  you will activate the connection to the measurement device:



Start Measure

The scale parameters (type, mode and interface) must be set when first measurement is taken.

 **The instructions for ViScan device must be followed when connecting to a PC!**

The measurement starts by clicking <Start Measure>. The relevant instructions or error messages will be displayed in the information window if a connection to ViScan cannot be established. This is most convenient, but is not supported by all scales.

There are two possible operating modes:

- **Standard mode:** start measurement at terminal of ViScan, PC is waiting for transmission of measured values.
- **Manual input:** if there is no connection between the PC and the ViScan, the readings can be copied from the terminal.

Following the successful transfer of readings this dialogue box will close automatically and the data entered in the health monitor. Click **<Save>** to save all data displayed to the database. The next reading can then be taken. If data is not saved it will be discarded when the next measurement is taken (e.g. in the event that a measurement is taken again because it is clearly incorrect) or on closing the index card.

[<top>](#)



[Overview about GMON modules](#)

general operation instruction, using goals values, other relevant parameters for health
[Index](#) of GMON program